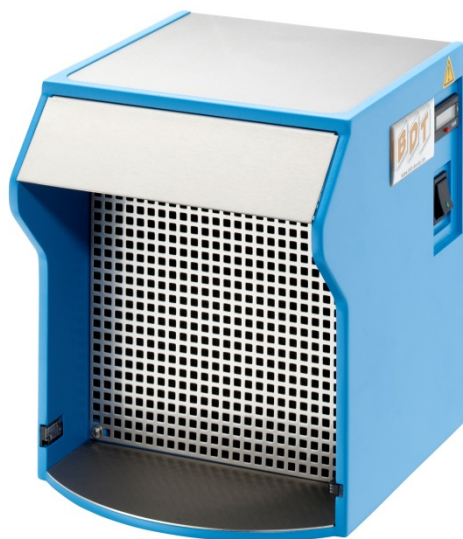
ABSAUGGERÄT

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EXTRACTOR

with activated charcoal filter

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Abbildung 6

Dear Customer

Thank you for choosing this high-quality product from BDT.

In order to achieve the best results with this product, it is important to carefully study and follow these instructions for use.

These written instructions cannot cover all possible situations that can show up during use. For this reason, our hotline is available to answer all questions or concerns that may arise.

Due to constant developments, we recommend, even when using the same products regularly, to study the enclosed current instructions for use or to refer to our website at www.bdt-dental.de.

Manufacturer

BDT GmbH • Sonnhalde 1 • 77749 Hohberg • Germany

Telephone +49 7808 914996-0

Fax +49 7808 914996-99

E-Mail info@bdt-dental.de

With reference to our quality standards

BDT ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products.

Responsibility for failures cannot be taken, as we have no influence on the processing on site.

Instruction for use

Extractor with activated charcoal filter

Artikelnr. 20192

1. Safety information

1.1 Correct usage

The suction unit is only intended for sucking out vapors when working with hazardous materials in the dental/orthodontic laboratory. The use of the unit for any other purpose is prohibited.

The use of the unit for its correct purpose also includes compliance with all the instructions contained in this handbook.

When a BDT product is finally no longer in use the disposal regulations of that particular country apply. BDT or the dental trade can be contacted to answer questions regarding the correct disposal of any specific product.

1.2 Danger symbols and markings

In the instructions for use, the following symbols are used to indicate sources of danger to the user:

Warning This symbol indicates direct danger to the life or health of the user. Disregarding this warning may involve a serious risk to health.

Caution This symbol indicates important safety instructions. Disregarding this warning may cause damage to the unit and/or other property.

1.1 Disposal

This symbol indicates that products marked with it should not be disposed of together with garbage of households. The legislator disallows commercial customers to return electronic waste via municipal collection points. Further information is available through BDT or the dental trade.

2. Description and usage

The suction unit helps to protect vapors, when working with hazardous materials in the dental/orthodontic laboratory. When using this suction unit, you avoid hazardous influences already when they develop right from the start. With the user friendly suction unit you provide a well-directed prevention against dangers through chemical materials.

Caution: Please note that optimal performance of this suction unit can only be assured if periodically maintained. Especially the filter cartridge has to be replaced at regular intervals.

Also follow national regulations and compliance with occupational limits of harmful substances.

3. Technical data

Mains connection	230 V	50/60 Hz
Rated power	100 W	
Dimensions (WxHxD)	225 x 260 x 260 mm	
Weight	6 kg	
Protection class	I	
Degree of protection	IP 22	
Operable temperature range	10 °C up to 40 °C (50 °F up to 104 °F)	
Use	indoor	
Height above sea level	max. 1500 m ü. NN	
Air humidity	10 % - 80 %, non-condensing	

4. CE- Declaration of conformity

Through the CE marking on the rating plate we confirm compliance with the EMV Directives of the EEC 89/336/EEC, Low-voltage Directive 73/23/EEC und CE-Marking Mandatory Directive 93/68/EEC.

In the event of any modification of the product without our approval this declaration will become invalid and the warranty will extinct.

5. Commissioning of the suction unit

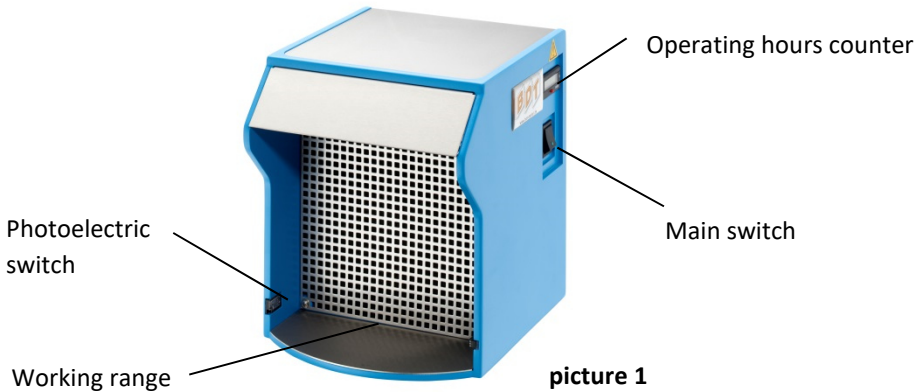
Should you notice any damage which may have occurred in transit, notify BDT Customer Service or the dealer you got the unit from without delay.

Before using the unit, ensure that the power supply voltage corresponds to the operating voltage stated on the rating plate on the back of the unit.

Before you use the device for the first time, remove the protective foil on the coupon plate. After connection to the mains (230 V) the unit is ready for operation. After turning the main switch the light is switched on automatically and the suction operation runs for about 5sec.

The suction operation is started automatically during works in the working area via photoelectric switch. On completion of the working procedure the suction operation is stopped after about 5sec. The operating hours counter is reset using the red button. After the suction unit has been switched on, the entire operating time is displayed for approx. 10 sec. Change the filter cartridge after 50 operating hours or once a year.

If any item lies in the working area, it will be identified by the photoelectric switch pin; the unit will not be switched off automatically

For safety reasons, it is not allowed to operate with an open flame in the working area (Fire hazard).

6. Care and maintenance

The unit is maintenance-free. If required, wipe case and working area with dry or moistened cloth. Do not use any aggressive detergents!

picture 2

7. Filter change

1. Disconnect the unit from the mains.
2. After releasing the 4 screws No.1 (pic.3) the protective grating and the preliminary filter can be removed. (pic.4)
The preliminary filter should be cleaned depending on the level of contamination.
3. After releasing the 4 screws no.2 (pic.5) the filter cartridge can be replaced.

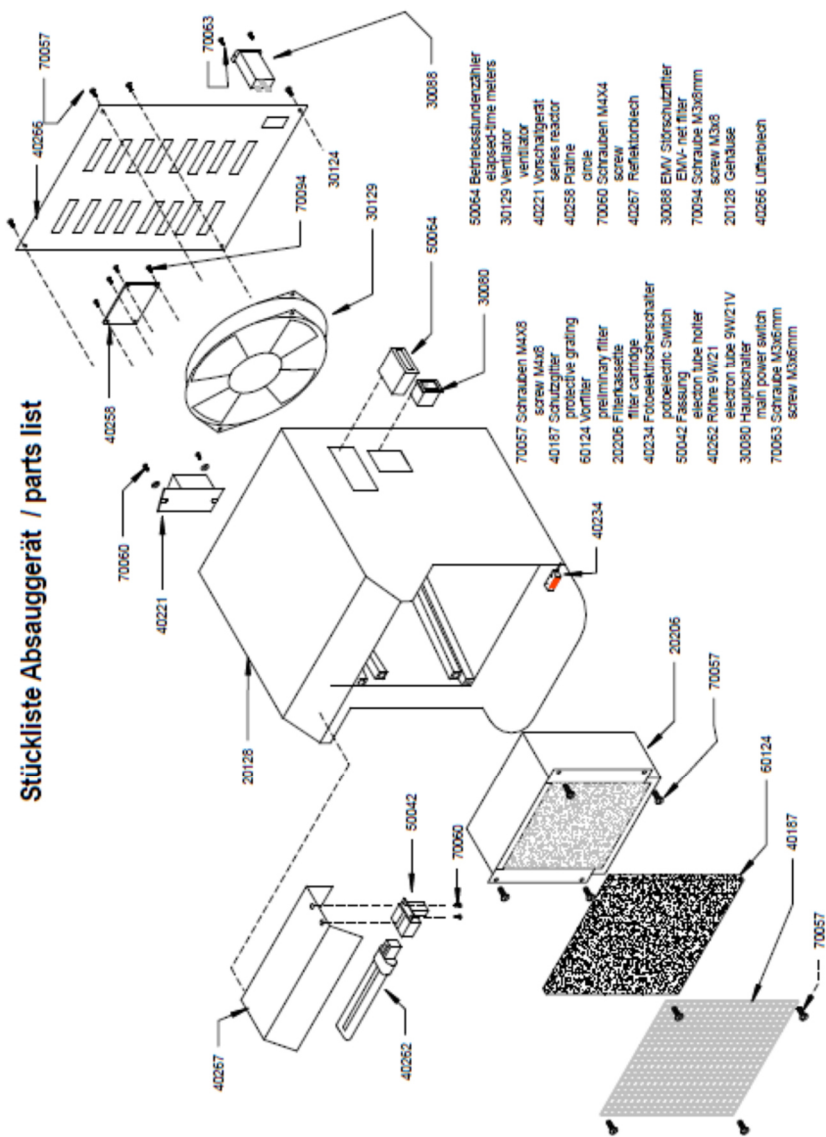
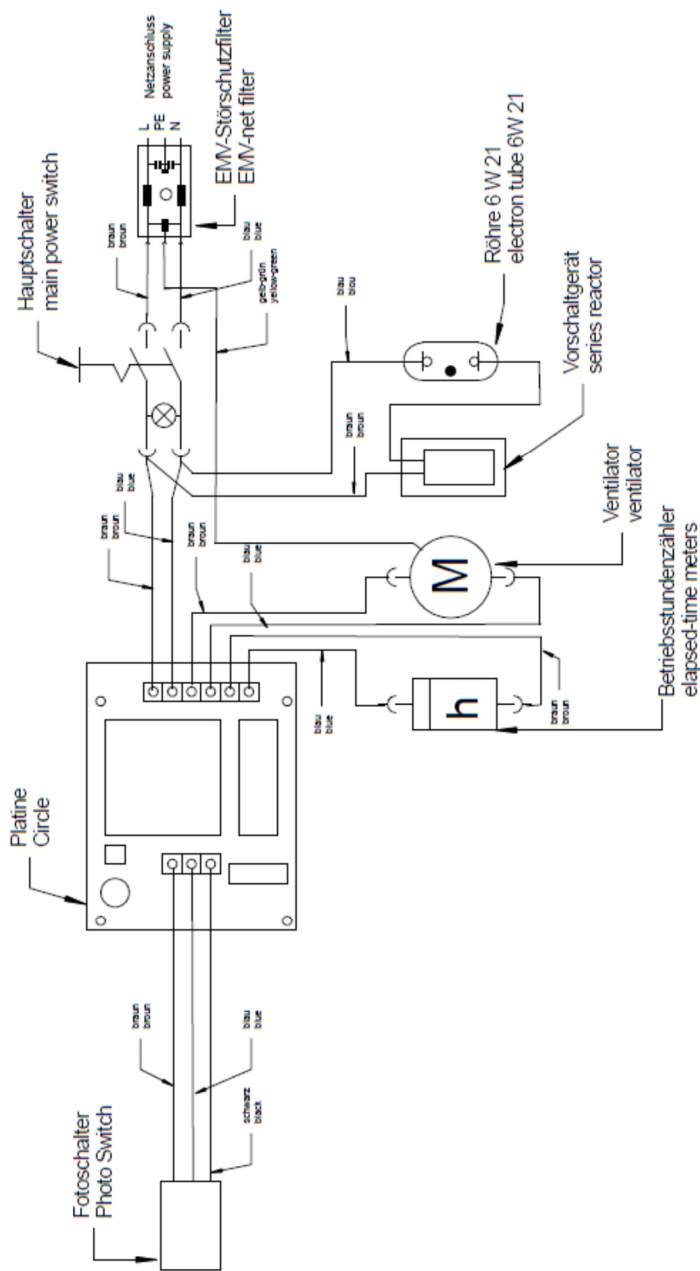
The filter cartridge must be changed after about 50 hours of operation.

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Gerätetechnik GmbH**
Seite 2 Empfänger BDT - Gesellschaft für Gerätetechnik GmbH, 77656 Offenburg
des Umweltschutzes GmbH