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Water Control with foot switch

For cold and hot water

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Instruction for use

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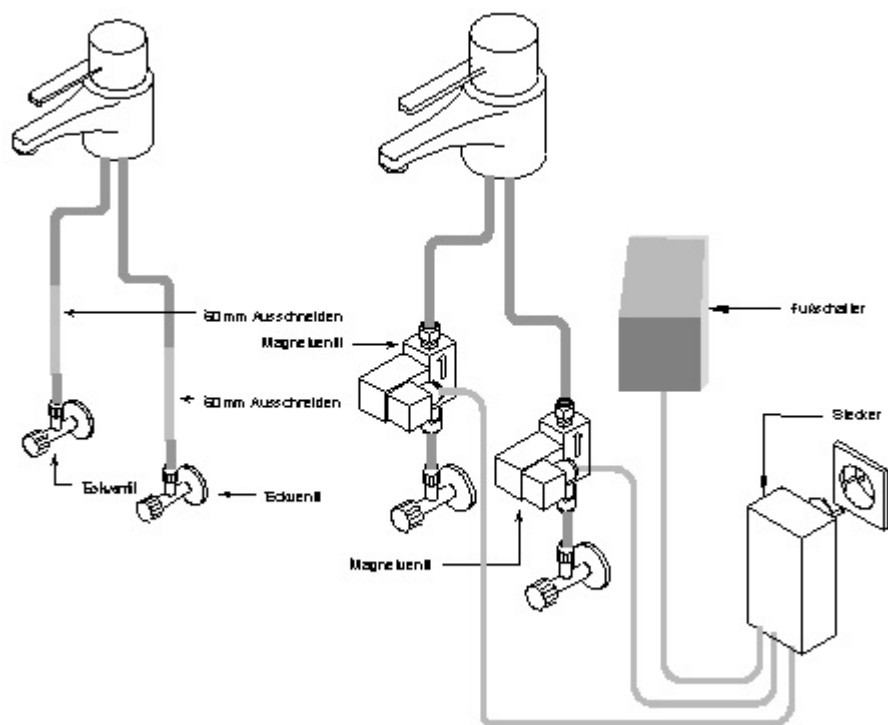
Dear Customer

Thank you for choosing this high-quality product from BDT.

In order to achieve the best results with this product, it is important to carefully study and follow these instructions for use.

These written instructions cannot cover all possible situations that can show up during use. For this reason, our hotline is available to answer all questions or concerns that may arise.

Due to constant developments, we recommend, even when using the same products regularly, to study the enclosed current instructions for use or to refer to our website at www.bdt-dental.de.

Manufacturer

BDT GmbH • Industriestr. 27 • 77656 Offenburg • Germany

Telephone +49 781/58 688

Fax +49 781/58 664

E-Mail info@bdt-dental.de

With reference to our quality standards

BDT ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products.

Responsibility for failures cannot be taken, as we have no influence on the processing on site.

Instruction for use

Water Control with foot switch

Artikelnr. 10026

1. Safety information

1.1 Correct usage

The water control is to be used to refer complained of water only on taps which are directly on the water pipe or a boiler

The use of the unit for its correct purpose also includes compliance with all the instructions contained in this handbook.

When a BDT product is finally no longer in use the disposal regulations of that particular country apply. BDT or the dental trade can be contacted to answer questions regarding the correct disposal of any specific product.

1.2 Danger symbols and markings

In the instructions for use, the following symbols are used to indicate sources of danger to the user:

Warning This symbol indicates direct danger to the life or health of the user. Disregarding this warning may involve a serious risk to health.

Caution This symbol indicates important safety instructions. Disregarding this warning may cause damage to the unit and/or other property.

1.1 Disposal

This symbol indicates that products marked with it should not be disposed of together with garbage of households. The legislator disallows commercial customers to return electronic waste via municipal collection points. Further information is available through BDT or the dental trade.

2. Description and usage

The water control with foot switches for cold and hot water is intended for installation in existing or new sink. By using the water control precious drinking water is saved and there are both hands free to work.

3. Technical data

Mains connection	230V	50/60Hz
Rated power	20 W	
Dimensions (WxHxD)	120x65x40 mm	
Weight	1,5 kg	
Protection class	I	
Degree of protection	IP65	
Operable temperature range	10°C up to 40°C (50°F up to 104°F)	
Use	indoor	
Height above sea level	max. 1500 m ü. NN	
Air humidity	10% - 80%, non-condensing	

4. CE- Declaration of conformity

Through the CE marking on the rating plate we confirm compliance with the EMV Directives of the EEC 89/336/EEC, Low-voltage Directive 73/23/EEC und CE-Marking Mandatory Directive 93/68/EEC.

In the event of any modification of the product without our approval this declaration will become invalid and the warranty will extinct.

5. Commissioning of the suction unit

Should you notice any damage which may have occurred in transit, notify BDT Customer Service or the dealer you got the unit from without delay.

Before using the unit, ensure that the power supply voltage corresponds to the operating voltage stated on the rating plate on the back of the unit.

The installation of water control should be performed only by qualified personnel

For installation of water control with foot switches for cold and hot water, close the angle valves of the water pipes.

Remove approximately 60 mm of lead water lines to the faucet and put the solenoid valves in the flow direction (arrows on the solenoid valve).

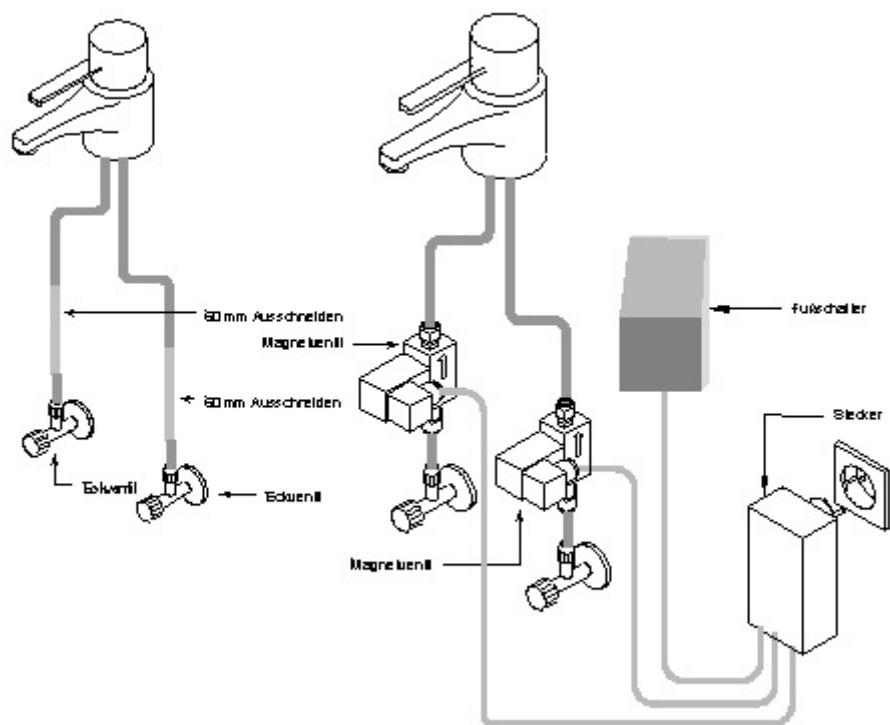
Mount the foot switch in the desired position.

Now put the housing into a grounded outlet.

Now open again the angle valves and the faucet. This must always remain open.

For water extractions press the foot switch.

The water control can also be operated at a boiler

6. Care and maintenance

The unit is maintenance-free