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---

## Dear Customer

Thank you for choosing this high-quality product from BDT.

In order to achieve the best results with this product, it is important to carefully study and follow these instructions for use.

These written instructions cannot cover all possible situations that can show up during use. For this reason, our hotline is available to answer all questions or concerns that may arise.

Due to constant developments, we recommend, even when using the same products regularly, to study the enclosed current instructions for use or to refer to our website at [www.bdt-dental.de](http://www.bdt-dental.de).

## Manufacturer

BDT GmbH • Sonnhalde 1 • 77749 Hohberg • Germany

Telephone +49 7808 914996-0

Fax +49 7808 914996-99

E-Mail [info@bdt-dental.de](mailto:info@bdt-dental.de)

## With reference to our quality standards

BDT ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the operation of the products.

Responsibility for failures cannot be taken, as we have no influence on the operation on site.

BDT cannot be held responsible for products produced using the equipment due poor operation by the user.

---

## Instruction for use

# Airblower and Airblower Silent

### 1. Safety information

#### 1. Intended use

The Airblower is exclusively designed to blow off dust from tooth models of dental - / orthodontic laboratories.

Any other use or use beyond this is deemed as improper use.

Intended use also includes observing the instruction manual.

When it comes to final decommissioning of BDT- device, please follow country specific regulations. To ensure proper disposal, please contact your local specialist trade partner or BDT.

#### 2. Danger symbols and markings

In the instructions for use, the following symbols are used to indicate sources of danger to the user:

**Warning** This symbol indicates direct danger to the life or health of the user. Disregarding this warning may involve a serious risk to health.

**Caution** This symbol indicates important safety instructions. Disregarding this warning may cause damage to the unit and/or other property.

#### 3. Disposal

This symbol indicates that products marked with it should not be disposed of together with garbage of households. The legislator disallows commercial customers to return electronic waste via municipal collection points. Further information is available through BDT or the dental trade.

---

## 2. Technical data / environmental conditions

Dimensions (WxHxD)	141 mm Ø 22 mm
Weight	0,1 Kg
Pressure max.	6 Bar

## 3. Installation of airblower

Before exchanging the airblower, ensure that no compressed air is left within the air hose. If necessary, compressed air supply must be interrupted.

Remove rubber nozzle and union nut from the compressed air connection of the air blower.

Now push the air hose through the rubber nozzle, followed by the union nut. Then press the compressed-air tube on connection piece of compressed air connection as far as possible. Screw union nut back onto compressed air connection and pull rubber nozzle over the compressed air connection.

## 4. Use of airblower

For adjustment of air volume, turn the adjusting screw with a screwdriver in required direction.

Now you can use the airblower by pressing the rubber part of airblower sideward with thumb.

## 5. Maintenance and care

The airblower may become untighten if soiling (like e.g. splinter) is inside compressed air hose. To regain full capacity of airblower again, please unassemble airblower and clean the sealing.

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