---

# Abzugsschrank ASMB430

## Extraction cabinet ASMB430

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---

## Dear Customer

Thank you for choosing this high-quality product from BDT.

In order to achieve the best results with this product, it is important to carefully study and follow these instructions for use.

These written instructions cannot cover all possible situations that can show up during use. For this reason, our hotline is available to answer all questions or concerns that may arise.

Due to constant developments, we recommend, even when using the same products regularly, to study the enclosed current instructions for use or to refer to our website at [www.bdt-dental.de](http://www.bdt-dental.de).

## Manufacturer

BDT GmbH • Sonnhalde 1 • 77749 Hohberg • Germany

Telephone +49 7808 914996-0

Fax +49 7808 914996-99

E-Mail [info@bdt-dental.de](mailto:info@bdt-dental.de)

## With reference to our quality standards

BDT ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products.

Responsibility for failures cannot be taken, as we have no influence on the processing on site.

---

# Instruction for use

## Extraction cabinet ASMB430

Artikelnr. 10022

### 1. Safety information

#### 1.1 Correct usage

The Extraction cabinet is designed exclusively for the storage and processing of hazardous materials in the dental / orthodontic laboratory.

The use of the unit for its correct purpose also includes compliance with all the instructions contained in this handbook.

When a BDT product is finally no longer in use the disposal regulations of that particular country apply. BDT or the dental trade can be contacted to answer questions regarding the correct disposal of any specific product.

#### 1.2 Danger symbols and markings

In the instructions for use, the following symbols are used to indicate sources of danger to the user:

**Warning** This symbol indicates direct danger to the life or health of the user. Disregarding this warning may involve a serious risk to health.

**Caution** This symbol indicates important safety instructions. Disregarding this warning may cause damage to the unit and/or other property.

#### 1.1 Disposal

This symbol indicates that products marked with it should not be disposed of together with garbage of households. The legislator disallows commercial customers to return electronic waste via municipal collection points. Further information is available through BDT or the dental trade.

---

## 2. Technical data

Mains connection	230 V	50/60 Hz
Rated power	150 W	
Dimensions (WxHxD)	1000 x 1250 x 500mm	
Weight	60 kg	
Protection class	I	
Degree of protection	IP 20	
flow volume	430 m <sup>3</sup> /h	
pressure	340 Pa	
Hose connection	3/4"	pressure max. 10 Bar
Discharge hose	Ø 22 mm	
Operable temperature range	10 °C up to 40 °C (50 °F up to 104 °F)	
Use	indoor	
Height above sea level	max. 1500 m ü. NN	
Air humidity	10 % - 80 %, non-condensing	

## 3. CE- Declaration of conformity

Through the CE marking on the rating plate we confirm compliance with the EMV Directives of the EEC 89/336/EEC, Low-voltage Directive 73/23/EEC und CE-Marking Mandatory Directive 93/68/EEC.

In the event of any modification of the product without our approval this declaration will become invalid and the warranty will extinct.

---


#### 4. Commissioning of the Extraction cabinet

Should you notice any damage which may have occurred in transit, notify BDT Customer Service or the dealer you got the unit from without delay.

Before using the unit, ensure that the power supply voltage corresponds to the operating voltage stated on the rating plate on the back of the unit.

After installing the extraction units connect the exhaust pipe. The diameter of the output is 100 mm. The maximum length of the exhaust pipe is 12 m and for each 90 ° bend must be 1 meter subtracted. Now connect the water inlet hose with the water supply 3/4" (max 10 bar). Afterwards the waste water hose must be attached to the waste system. Now connect the device to the mains (230 V) and press the power switch. The left main switch turns the light on. The right main switch turns the motor on. By the speed controller of the motor can be regulated steplessly.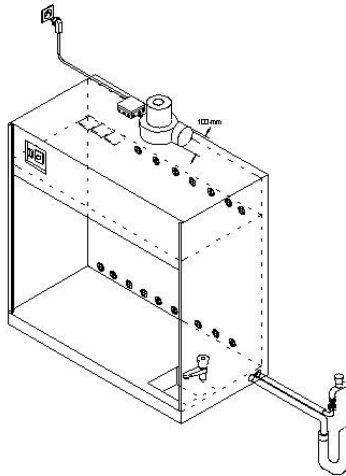

#### 5. Care and maintenance

The unit is maintenance-free. If required, wipe case and working area with dry or moistened cloth.